OrthoWedge
+ OrthoWedge Pediatric

DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Toll-free 800.999.8866
Phone 304.522.4883
Fax 304.522.0037
www.darcointernational.com

DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07 92 28-0
Fax +49 (0) 88 07 92 28-22
info@darco.de, www.darco.de

Made in China

USA/GB Indications

- > Use after surgeries to correct hallux valgus, hammer toes, Tailor's bunions and other osteotomies
- > Treatment of diabetic ulcerations (forefoot)

Contraindication:

- > Open fractures

Technical data

Sizes:
5 Adult sizes (OrthoWedge):
XS (33,5 – 37), S (37,5 – 39,5), M (40 – 41,5), L (42,0 – 44,0), XL (44,5 – 47,0)

1 Children size (OrthoWedge Pediatric):
Universal (up to size 31)

The shoe fits right and left equally.

Material:
Outsole: TPR
Insole: EVA
Upper material: Polyester
Hook and loop fastener straps: Nylon

Maintenance:
The product does not require maintenance.

Construction features:
The inner sole can be removed and replaced by the PegAssist® Insole.

Recommended Add-ons:
TwinShoe®, Balance shoe
PegAssist® Insole, individually customizable insole for targeted off-loading

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OrthoWedge

USA/GB Instructions for use

OrthoWedge
Shoe to reduce pressure on the forefoot

- Open the shoe (A).
- Insert the foot into the opened shoe.
- Make sure that the foot is positioned securely in the shoe.
- Close the shoe with the hook and loop fasteners (A). So that the fit is snug, but not too tight.
- Only roll as far as the inner edges (B) when walking. Do not roll-off as far as to the outer edges (C). Otherwise it may negatively influence the final outcome of surgery.
- When worn for longer periods, it is recommended that the height of the other foot is adapted to match (TwinShoe®). The shoe should not be worn for longer than 6 weeks.

Caution

OrthoWedge

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. If the product is handled incorrectly, its function and optimum protection can no longer be guaranteed. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. It is advisable to wear dressings or socks with the shoe. We recommend that you do not drive a vehicle while you are wearing this aid. Use extreme caution when climbing stairs (e.g. also stepping up onto curbs) and walking on uneven surfaces. Walk more slowly than usual. This aid is intended for single-patient use only.

Cleaning instructions

OrthoWedge

- Hand washing:**
 - When the shoe is slightly dirty, clean it with a damp cloth or a soft brush.
 - The shoe may not tumbled dry or dried on the heater. Allow the shoe to air dry at moderate temperatures.

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