

SlimLine™

+ Pediatric SlimLine™



+ DARCO

Instructions for use



SlimLine™



Pediatric SlimLine™

SlimLine™ / Pediatric SlimLine™

Cast Boot

1. Open the boot by means of the three / two hook and loop straps (A).
2. Enter foot into open boot.
3. Please ensure that the foot is securely positioned in the boot and that the heel area of the foot is as far back as possible in the boot.
4. Close boot with the hook and loop straps (A). The foot should be placed comfortably in the boot, not too loose and not too tight.

Optional:

The inner sole can be removed / replaced by a healthcare specialist.

Indications

- > Fracture and cast care

Caution

SlimLine™ / Pediatric SlimLine™ cast boot has been prescribed by your physician to serve a particular purpose. Its design allows for the most rapid healing of fractures and recovery after surgery with unlimited mobility at the same time. Walking with this boot is more difficult than walking with conventional footwear; please read the following precautions. Do not attempt to drive while wearing this boot. Use extreme caution when climbing stairs, stepping up onto curbs and walking on uneven surfaces. Walk more slowly than usual. You are not wearing a normal street boot and more clearance is required for the wedge sole. Excessive weight bearing (edematous patients) may cause a breakdown of the boot's functional sole. In case the sole must be fixed due to extensive wearing, please contact your physician or orthopedic technician. Recommendation to wear with bandages or socks. If the product is improperly handled its function and optimal safety can no longer be guaranteed. This aid is intended for single-patient use only.

Technical data

Sizes:

SlimLine™: XS (up to 4), S (4.5 – 6), M (6.5 – 8), L (8.5 – 10), XL (10.5 – 12.5)

Pediatric SlimLine™: S (up to 4.5), M (5.5 – 9), L (10 – 12.5)

The boot is a universal fit – left or right

Colours:

Adult model (SlimLine™): Black

Children's model (Pediatric SlimLine™): Blue

Material:

Outsole: TPR

Insole: EVA

Upper material: EVA / Polyester

Hook and loop: Nylon

Maintenance:

The product does not require maintenance.

Construction features for adult and children's model:

The upper part consists of a breathable EVA / polyester material.

The rubber sole is made of TRP with shock absorbing effect. The sole profile enables an easier rolling-off movement.

Cleaning instructions

Hand washing:

- > When the boot is slightly dirty, clean it with a damp cloth or a soft brush.
- > Do not tumble dry or dry using heat. Allow the boot to air dry at moderate temperatures.



DARCO International, Inc.
810 Memorial Blvd.
Huntington, WV 25701, USA
Toll-free 800.999.8866
Phone 304.522.4883
Fax 304.522.0037
www.darcointernational.com



DARCO (Europe) GmbH
Gewerbegebiet 18
82399 Raisting, Germany
Telefon +49 (0) 88 07.92 28-0 | Fax -22
info@darco.de, www.darco.de

Made in China



190718